**631**

ART. 114.631

**GranberG**<sup>®</sup>**631**

ART. 114.631

**GranberG**<sup>®</sup>

AQL 1.5

Available sizes: S (6-7), M (7-8), L (8-9), XL (9-10)  
 Tilgjengelige størrelser: S (6-7), M (7-8), L (8-9), XL (9-10)  
 Tillgängliga storlekar: S (6-7), M (7-8), L (8-9), XL (9-10)  
 Saatavilla olevat koot: S (6-7), M (7-8), L (8-9), XL (9-10)  
 Dostępne rozmiary: S (6-7), M (7-8), L (8-9), XL (9-10)

For minimal risks only

**EN: USER MANUAL****NO: BRUKERVEILEDNING****SV: BRUKSANVISNING****FI: KÄYTTÖOHJE****PL: INSTRUKCJA OBSŁUGI****GranberG**<sup>®</sup>**Head office:**

GRANBERG AS

Bjoavegen 1442, NO-5584 Bjoa, Norway  
 Tel +47 53 77 53 00, e-mail post@granberg.no

**Swedish office / European warehouse:**

GRANBERG SVERIGE AB

Schubergvägen 20, 311 74 Falkenberg, Sweden  
 Tel +46 (0)346 124 25, e-mail post@granberg-ab.se

**granberggloves.com****ENGLISH**Single-Use Gloves Magic Touch<sup>®</sup>. Soft Nitrile<sup>™</sup>, powder-free. Blue colour.**Medical Device Regulation**

- I. This product is classified as a Class I medical device and meets the provisions of the Medical Device Regulation (EU) 2017/745.
- II. This product complies with European Standards EN 455-1:2000, EN 455-2:2015, EN 455-3:2015, and EN 455-4:2009.

**EU Type-Examination**

This product is classed as Category I Personal Protective Equipment (PPE) according to European PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standard EN 420:2003+A1:2009.

Gloves in compliance with the Regulation (EC) 1935/2004 and according to Commission Regulation (EU) No. 10/2011 suitable for handling all types of foods for short period of time.

**Performance and Limitation of Use**

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

Before usage, inspect the gloves for any defect or imperfections.

Components used in glove manufacturing may cause allergic reactions in some users. If allergic reactions occur, seek for medical advice immediately.

**Product Instruction for Use**

- I. Usage – For Single Use only. If re-used, the risk of contamination and infection increases due to improper cleaning processes; and increased risk of holes and tear during re-use due to weakening of gloves by cleaning processes.
- II. Sizing – Select the right size glove for your hand.
- III. Donning – Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to a get a good fit. Don the other glove by the same procedure.
- IV. Inspection – Punctures or tears may occur after donning. Inspect each glove after donning, and immediately discontinue use if found damaged.
- V. During Use - Gloves polluted by blood or other organic fluids must be immediately removed as medical waste. In case of pricking or tearing, user should immediately change gloves.
- VI. In case of feeling looseness on fingers, user should change gloves.
- VII. Doffing – Hold glove bead and pull toward the finger until the glove come off.

**Storage Conditions**

Store in a cool and dry place. Opened boxes should be kept away from fluorescent and sunlight.

Ideally stored in dry condition.

When stored as recommended, the gloves will not suffer any changes in chemical properties for up to three years from date of manufacture. Service life cannot be specified and depends on application and responsibility of user to ascertain suitability of the glove for its intended use.

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